L O L A & L Y K K E ®

Lola&Lykke® Core Relief Pregnancy Support Belt

Instruction for use

(GB)	Instruction for use	2	(FR)	Mode d'emploi	38
(CZ)	Návod k použití	8	(IT)	Istruzioni per l'uso	44
(DA)	Brugsanvisning	14	(LT)	Naudotojo vadovas	50
(DE)	Gebrauchsanleitung	20	(NL)	Gebruiksaanwijzing	56
(ET)	Kasutusjuhend	26	(NO)	Bruksanvisning	62
(FI)	Käyttöohjeet	32	(SE)	Bruksanvisning	68
			(AR)	تعليمات الاستخدام	74

LOT see packaging

Lola&Lykke Oy
Konepajankuja 1
00510 Helsinki
Finland

EC **REP**

INSTRUCTIONS FOR USE

Lola&Lykke® Core Relief Pregnancy Support Belt

Dear Customer, thank you for choosing our support belt!

The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the "Usage and safety instructions" - before using this product for the first time. You can download the Instructions for Use from our website, at www.lolalykke.com. We will be glad to answer any questions and to receive any suggestions you might have.

Your Lola&Lykke team

Please observe and retain the instructions for use and the packaging.
Made in China.

FIELD OF APPLICATION

These instructions for use apply to the following products:

TJ-CJ-MBB-S - Lola&Lykke support belt, size S

TJ-CJ-MBB-M - Lola&Lykke support belt, size M

TJ-CJ-MBB-L - Lola&Lykke support belt, size L

TJ-CJ-MBB-XL - Lola&Lykke support belt, size XL

PRODUCT CONTENT

1 Lola&Lykke support belt in the selected size (S, M, L, or XL).

1 Hot/Cold Therapy Pack.

1 instruction manual.

PURPOSE AND INTENDED USE

The Lola&Lykke Core Relief support belt is an orthopaedic belt, consisting of a belt with fully adjustable hook and loop fastener, elastic compression straps as well as stabilising back rods. The belt is designed for short-term use around the hips and lower back. It is intended to be worn by women during pregnancy to ease pain in the back, hips, or pelvic girdle and to take the weight of the baby belly.

AREAS OF APPLICATION

The Lola&Lykke support belt is for expectant mothers with back pain or pelvic girdle pain.

FUNCTIONALITY

The Lola&Lykke support belt supports the stomach and alleviates back pain by taking some of the burden off the muscles of the back and pelvic floor. The fully adjustable hook and loop fastener means that you can adjust the belt to suit individual requirements. This means that the support belt can be worn throughout the entire pregnancy. Using the two additional elastic straps, the supportive effect can be adjusted to fit personal requirements for comfort and stability.

CORE SUPPORT

EASES PRESSURE





EASES BACK PAIN

REDUCES STRETCH MARKS

USER INSTRUCTIONS

It is important that you use the right size of Lola&Lykke support belt and that it is positioned correctly, to achieve the best level of support. The graphics and product images on the packaging indicate the correct sizing and the correct positioning of the belt. When doing this, follow these steps: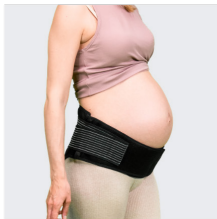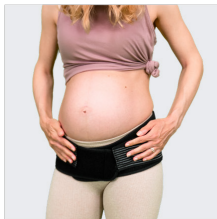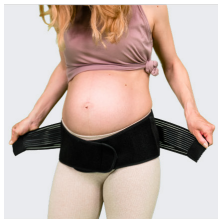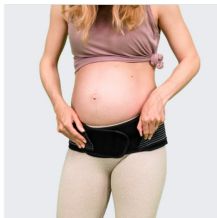

1. Open the support belt by undoing the hook and loop fastener.
2. Place the belt with the large surface level with the lower back and fasten the belt by putting the right side over the left side under the stomach.
3. Fasten the hook and loop around the waist, under your belly, so that it feels comfortable.
4. Take both elastic bands into your hands simultaneously and tighten them as evenly as possible, so that there are no folds in the fabric that could cause a pressure sensation.
5. Fix the elastic straps onto the loop strips, to give you a pleasant supportive effect.
6. If you feel that the support belt is too tight or too loose, repeat steps 1 - 5 and change the tension of the elastic straps. There is an application video on our website to show you how to position the belt correctly.

SAFETY INSTRUCTIONS

The support belt should only be worn when standing or walking and not during breaks, when carrying out sedentary tasks or when sleeping. If, while wearing the belt, you should start to feel unwell (dizziness, high pressure) then please undo the support belt immediately.

Side effects could include skin irritation such as redness, swelling or itching due to a build-up of heat in the area of the support belt. If this happens, take off the support belt immediately.

The support belt must not be worn if you have open wounds in the stomach, pelvic and back area. Before use, check the support belt to make sure that it is intact. Do not use the support belt if, for example, the rods on the back are sticking out, to avoid any risk of injury.

If the belt is pressing too hard or slips, then you might need a different size. See the measurement chart/sizing on the packaging.

In case of a known hypersensitivity and allergic reactions to one of the material components mentioned, you should stop using the support belt or only use it after having consulted your physician.

If the symptoms persist or worsen, please seek advice from a doctor.

All serious incidents occurring in connection with this product must be reported to the manufacturer immediately.

HOT/COLD THERAPY PACK USER INSTRUCTIONS

TO USE COLD

Store in freezer, remove and place in the pocket or other additional cover. Apply to affected area. A minimum of 2 hours is required in the freezer to obtain the best results.

DO NOT USE DIRECTLY ON SKIN STRAIGHT FROM THE FREEZER.

TO USE HOT

Hot water use:

1. Boil enough water to cover the pack completely and remove from heat.
2. Place the pack in the water for about 7 minutes .
3. Leave in container for 10 minutes if pack is frozen.
4. Remove from water using a tong or similar and place in the pocket.

Microwave use:

1. Be cautious as microwaves vary in power and you will need to find the correct amount of time to heat the pack.
2. Flatten the pack to ensure the gel is evenly distributed.
3. Place the pack in the microwave oven and set timer for max. 30 seconds and then allow to stand for 30 seconds.
4. If more heat is required, heat in 10 second increments with equal standing time until desired temperature is achieved.
5. Remove from the microwave using a tong or similar and place in pocket.

NOTE: Medium power is recommended for the microwave. Do not heat in microwave oven over 1400 WATT. DO NOT USE DIRECTLY ON SKIN STRAIGHT FROM THE MICROWAVE.

CAUTION

Do not overheat the pack in the microwave beyond the recommended time as this may cause leakage of contents. If contents are exposed to the skin or eyes, flush with water immediately. The gel is non-toxic. If irritation or redness persists, consult a physician. The gel is inedible.

SAFEKEEPING, SERVICE LIFE, STORAGE, AND TRANSPORT

With normal usage, the product should be replaced after 3 years.

CLEANING AND DISPOSAL INSTRUCTIONS

The Lola&Lykke support belt is only suitable for hand wash and should be dried in open air without wringing or twisting. To prevent damage to the support belt, the use of chemical cleaning agents and ironing the support belt is not recommended. Please observe the cleaning and care instructions on the product to increase the service life of your belt. The Lola&Lykke support belt can be disposed of along with the household waste. Please observe the regional disposal regulations in the process.

MATERIAL COMPOSITION

The Lola&Lykke pregnancy support belts are composed of:
Shell: 95% Viscose (Bamboo) 5% Elastane
Core: 50% Polyester 50% Nylon
Vertical back support ribs (PP)

EXPLANATIONS OF THE SYMBOLS USED

Hand wash cold

Manufacturer

Do not bleach

Caution!

Do not tumble dry

Medical Device

Do not iron

Batch number

Do not dry clean

Observe instructions for use

For questions, please contact: info@lolalykke.com

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